



Geberit AquaClean Mera CARE

Shower Chair / Equipment Compatibility Report

3rd edition

Author – Simon Thomas

Updated – May 2021

Index

Pages.

- 4 6 Product overview
- 7-22 Shower chairs
- 23 32 Tilt in Space shower chairs
- 33 39 Paediatric equipment
- 40 46 Bariatric equipment
- 47 53 Toilet lifters
- 54 60 Miscellaneous Equipment
- 61 63 Product details & Manufacturer contact details
- How to access shower chair mode

Rimless ceramic – TurboFlush

App control

Wireless Opto remote

User remote control

Self servicing function

5 Year FREE Warranty

YouTube video support



Constant warm water

Industry leading 1 minute Wash cycle

Whirlspray Shower Technology

Oscillating Wash

Adjustable wash arm positioning

5 programable user settings

Bluetooth Technology






Compatibility

With an estimated 50% of wash dry toilets provided under a DFG grant being used with a type of shower chair or assistive equipment, compatibility is essential.

This report looks at the wide range of equipment available on the healthcare market that works satisfactorily with the Geberit AquaClean Mera <u>Care</u> wash dry toilet.

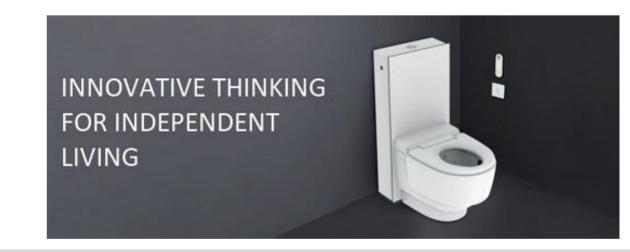





Shower Chairs

Attendant or self propel, there is a wide range of upright shower chairs available from several manufacturers.

Osprey Healthcare, Invacare, Freeway (Prism Medical), Aidservice, Care & Independence and Etac all have a range of upright shower chairs that are fully compatible with the Geberit AquaClean Mera **CARE** shower toilet.













Attendant Sani Commode

- Stainless Steel (Rust Resistant)
- Adjustable Seat Height 19"-22"
- Flexi Seat, Horseshoe Seat,
- Toilet Seat, Various apertures
- Detachable Foldaway Footrest
- Foldaway Padded Arms
- Compatible with Washer/Dryer toilets
- 30 Stone SWL







Self Propel Sani Commode

- Stainless Steel (Rust Resistant)
- Adjustable Seat Height 19"-22"
- Flexi Seat, Horseshoe Seat,
- Toilet Seat, Various apertures
- Detachable Foldaway Footrest
- Foldaway Padded Arms
- Compatible with Washer/Dryer toilets
- 30 Stone SWL








Bath Mobile Shower Toilet Chair

- Easily dismantled
- Rust proof
- Flip away arms
- 130kg Safe working load
- Range of accessories

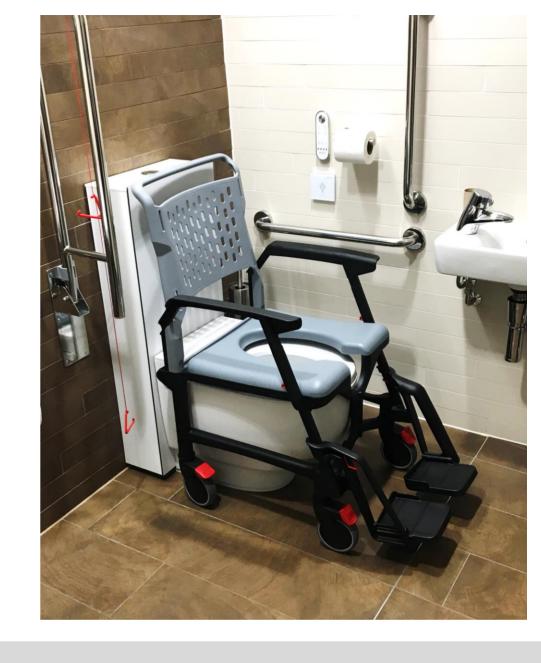





Care & Independence

Options 7710C Shower Chair

- Adjustable height from wheel to seat
- Different options of footrests detachable wheelchair style footrest or a one piece footrest
- Four armrest options
- Can be used over a toilet or with a commode pan
- Safe Working Load of up to 200kg
- Different wheel options available transit, attended and self propelled
- 15 year warranty on frame







Care & Independence

Options 7710C Shower Chair Self Propel

- Adjustable height from wheel to seat
- Different options of footrests detachable wheelchair style footrest or a one piece footrest
- Four armrest options
- Safe Working Load of up to 200kg
- Different wheel options available transit, attended and self propelled
- 15 year warranty on frame
- Padded seat options with skirts available in horse shoe or full aperture





Freeway (Prism Medical)

Freeway T40 Auto

Key Features

• The Freeway T40 / T70 AUTO is for use with automatic toilets. Features pram handle backrest and seat with skirt as standard.

Dimensions

- The Freeway T40 Shower Chair range is available in three standard sizes, based on seat width:
- 'A' 540mm or 21"
- 'B' 490mm or 19"
- 'C' 440mm or 17"







Freeway (Prism Medical)

Freeway T70 Auto

Key Features

 The Freeway T40 / T70 AUTO is for use with automatic toilets. Features pram handle backrest and seat with skirt as standard.

Dimensions

- The Freeway T40 Shower Chair range is available in three standard sizes, based on seat width:
- 'A' 540mm or 21"
- 'B' 490mm or 19"
- 'C' 440mm or 17"







Wealden Rehab

The Raz AP GEB Shower Chair

- Attendant Propelled
- Stainless steel height adjustable frame
- 25 stone weight limit
- Seat depth 400 or 480mm
- Overall length with footplate 860mm
- 5 Year warranty
- Tension adjustable back rest







Freeway (Prism Medical)

Freeway T60A / T60B

- Safe Working Load (SWL): 200kg (30 stone)
- Available as a shower, toilet or commode chair
- Perfect for use in tight spaces
- · Anti-tip bars fitted to the rear as standard
- Pliable backrest moulds to the user while providing excellent support
- FREECOTE®, antibacterial and antimicrobial coating.
- 19" or 21" seat widths only as standard







Invacare

Aquatec Ocean shower chair

- Stainless steel frame
- Machine washable back rest
- Adjustable height seat settings
- Horse shoe Seat
- Fold away footplates
- Maximum user weight 180kgs







Flyta Active shower chair

- Stainless steel frame
- 17" or 19" Seat widths
- 15 Year warranty
- Toilet, Horseshoe, or full aperture Seat options available
- 24" Self propel wheels
- Webbed calf rest
- 18 stone user weight limit







Wealden Rehab

The Raz SP GEB Shower Chair

- Attendant Propelled
- Stainless steel height adjustable frame
- 25 stone weight limit
- Seat depth 400 or 480mm
- Overall length with footplate 860mm
- 5 Year warranty
- Tension adjustable back rest








ETAC

Etac Clean Shower Commode Chair

- Unique foot support
- Swing away arm rests
- 130kg Safe working load
- Range of accessories

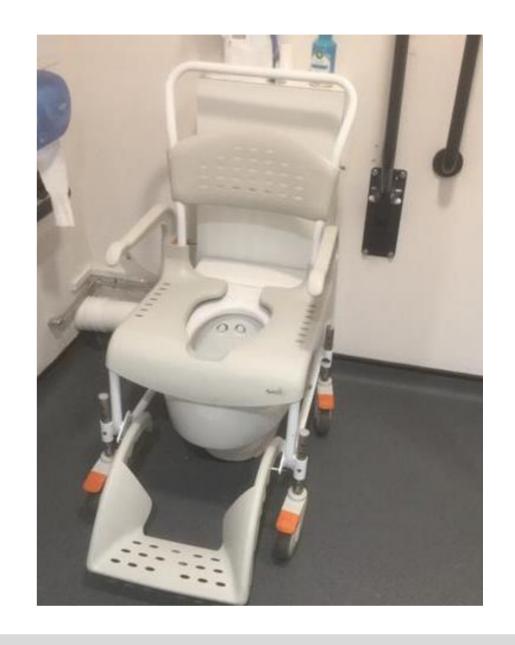





Drive Devilbiss

Aston Heavy Duty Shower Commode

- Seat width 56cm (22")
- Seat depth 43cm (17")
- Seat height 51 56cm (20" 22")
- Tool-free height and angle adjustable footplates
- Simple e-clip height adjustment to suit different toilet heights
- Armrests flip up for side transfers
- Rust proof aluminium frame
- MAXIMUM WEIGHT CAPACITY 260kg (40st)







Smit Care

SmitCare Apex Mobile Commode (SAC360)

Key features

- Four braked castors,
- Removable arms and swing-away, lift off footrests.
- Seat height 500mm (19.5")
- Seat size 435 x 460mm (17 x 18")
- Castors 130mm (5") diameter.
- 30 stone (190kg) max user weight limit

*NB. The products are not stainless steel and may rust. Geberit recommend use only on a short term basis / emergency use







<u>Tips</u>

Always specify that the shower chair is for use with the Mera
CARE shower toilet at time of ordering

AquaMaster
Range install
toilet at
430mm seat
height

All standard
Freeway
"Auto" Chairs
- install toilet
@465mm
seat height

Care & Independence

self propel –
install at 435mm
CERAMIC height and shower chair on its 2nd hole down to achieve the lowest shower chair seat height possible



Tilt In Space Shower Chairs

Tilt in Space shower chairs have gained in popularity. Offering safe & secure support for both toileting and showering.

Osprey Healthcare, Freeway (Prism Medical), Care and Independence, Chiltern Invadex all have a range of upright shower chairs that are compatible with the Geberit AquaClean Mera **CARE** shower toilet.





Care & Independence

Adult Tilt in Space Chair

- Adjust anywhere between upright and 45° to best suit your requirements
- Accessories include harness, thoracic support & hip straps
- Frame is made from stainless steel which helps with recycling
- Thoracic lateral support and lateral leg support available
- Safe Working Load of 150kg
- 15 year warranty on frame
- Padded seat options with skirts available in horse shoe or full aperture





981 Tilt in Space Chair

Key Features

- Easy to Tilt
- Adjustable Headrest
- Adjustable Footrest
- Supportive Mesh
- Optional Mesh Aperture
- Stainless Steel (Rust resistant)

SLIDE 25

COMPATIBILITY REPORT

Adjustable Seat Height





Invacare

Ocean VIP Ergo

- Currently the only TIS shower chair that works with a side exit soil pipe
- Stainless steel frame
- -5° to 40° seat tilt angle
- 480mm Seat width
- Modular Design
- Tilt in space
- Height Adjustable 510mm 610mm





Etac

The Swift mobile TILT

- Unique tilt action
- Adjustable backrest
- Height adjustable
- Head support
- Head Support
- Tool free adjustments







R82

The Heron manual TILT In Space

- Unique tilt action
- Adjustable backrest
- Height adjustable
- Head support
- Head Support
- Tool free adjustments







Freeway (Prism Medical)

T80 Geberit Mera CARE (Not AUTO)

- Easy to Tilt
- Removable Headrest
- Adjustable Footrest
- Freecote protective frame covering
- Wide range of positioning accessories







Chiltern Invadex

Aqua Master Tilt in Space Shower Commode Chair

A16G - 18" seat width

A17G - 21" seat width

- Full range of aperture, horseshoe and flat padded seats available
- Option for bespoke seating, including pressure relief foam
- Armrest, footrest and backrest options
- Extensive range of accessories available
- Dual gas rams to ensure smooth reliable operation
- Corrosion resistant stainless steel and nylon braked castors Directional locking option available





Wealden Rehab

The Raz AT Shower Chair

- Stainless steel chassis for durability
- Height adjustable seat
- Tension adjustable backrest supports posture and provides lateral support
- Tilt-in-Space to 40° excellent for pressure management and comfort
- Flip-up armrests for excellent access
- IPAS Seat System adjust the seat depth to achieve optimum aperture positioning







Tips

Freeway T80 Geberit Mera Care - install toilet at 465mm seat height.

Always specify with Freeway to be used with Mera CARE shower toilet

AquaMaster
Tilt in Space
install toilet at
435mm seat
height

Care &
Independence
Adult Tilt In
Space is height
adjustable to
suit any Gebert
seat height

Osprey
Healthcare
TIS is height
adjustable to
match Geberit
seat heights

Always specify
Tilt in space
shower chair is
for use with a
shower toilet
when ordering



Paediatrics

Paediatric equipment has rapidly improved in recent years, offering greater support and helping to promote independence in a safe environment

Osprey Healthcare, Freeway (Prism Medical), Rifton and R82 are some of the market leaders in Paediatric shower chairs.

Other more basic equipment is also available via some high street stores and internet sellers.





Freeway (Prism Medical)

T90 Paediatric Self Propel Shower Chair

- 365MM seat width
- Available as assistant or self propelled models
- Integral horizontal push handled backrest
- Pan sliders attach to the frame and are easily removable without tools
- Supplied as standard with straight paediatric armrests and small horseshoe seat
- Different height armrest sockets
- Different positions for armrest sockets
- Clearance on this frame is now 475mm Install Geberit @ 465mm





Freeway (Prism Medical)

T80 Paediatric Reclining Shower Chair

- Easy to Tilt
- Removable Headrest
- Adjustable Footrest
- Freecote protective frame covering
- Wide range of positioning accessories





Jiraffe (Rifton)

The HTS

- Three seat sizes all work with Mera **CARE**
- Tilt in Space
- Unique seat pad design
- Load bearing footplate
- Thoracic Supports
- Height Adjustable
- Seat depth Adjustable







R82

Flamingo

- Four different sizes
- Back angle adjustment
- Height adjustable
- Tilt in Space
- Adjustable footplates
- Side Supports
- Height Adjustable
- Head Support







Argos / Amazon / Online

Children's toilet seat with step

- Quick and affordable
- Overnight delivery
- Ideal for young children and individuals with Dwarfism
- Available in a wide range of colours





Tips

Rifton HTS & R82
Flamingo work best
with seat up –
contact Geberit
area manager at
time of install for
guidance

Children's toilet seat with step – Install Geberit at 430mm Install Geberit at 465mm for FreewayT80 Paediatric

All sizes of the Rifton HTS work over a Mera
CARE

The Smallest Flamingo does not work over a Mera CARE





Bariatrics

With an increase in awareness of the needs of larger people and the adaptations needed for existing equipment to support a much larger body weight there has never been such a greater focus. Equipment is now available to suit all Plus size, Mediatric and Bariatric users.

Equipment is now on the market capable of supporting users up to 85 stone






Osprey Healthcare

The Java Bench

- Choice of Toilet or Horseshoe seat
- Skirt Included for Geberit compatibility
- Safe working load of 85 stone
- Slight incline on the seat to aid with standing & comfort
- Also compatible with other wash/ dry and standard toilets
- 19" or 21" seat height
- Choice of 2 leg styles, bolt down or rubber ferrules.
- 15 Year frame warranty

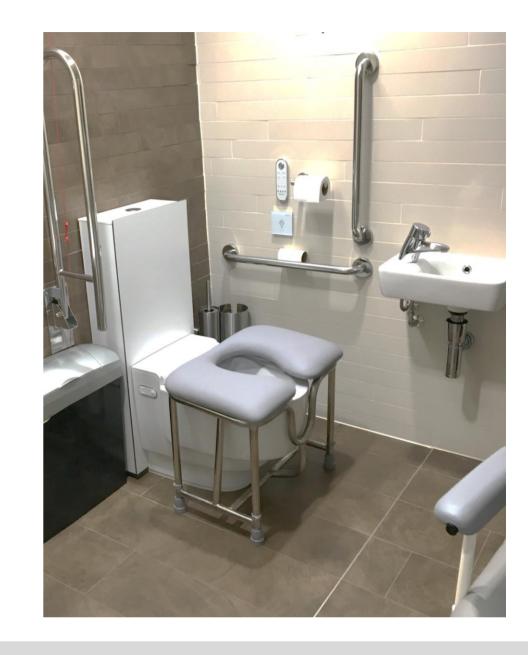





Chiltern Invadex

A02C Bariatric Bench

- 55 Stone safe working load
- 24" Seat width
- Horseshoe aperture with skirt
- Optional extra arm rests.







Langhams

Gordon Ellis Seat & Frame Horse Shoe

- Comfortable moulded plastic toilet seat
- · Seat easily clips on and off the frame for cleaning
- 30 Stone weight limit







Langhams

Gordon Ellis Seat & Frame Full Aperture

- Comfortable moulded plastic toilet seat
- · Seat easily clips on and off the frame for cleaning
- 30 Stone weight limit





Aidapt Sussex Bariatric Commode

- Vinyl padded over seat supplied for discreetness when not in use
- 7.5 litre commode bucket and lid provided as standard
- The Sussex commode ideal for users with limited mobility
- Can be used as a standalone commode or can be positioned over most WC suites
- Height adjustable legs and support arms as standard
- Maximum user weight limit 254kg (40 stones)







<u>Tips</u>

Removal of Seat gives better clearance for the "Shelf" and allows for full functional use of the wash and dry functions

Osprey seat height of 19" requires a Geberit seat height of 445mm.

Langham
Multi frame
also available
with a big john
seat
attachment

Removal of toilet seat requires a Free of charge visit from a Geberit area manager





Toilet Seat Raisers

Toilet seat raisers have been in circulation for many years and have become more popular in recent times due to an increase in manufacturers reducing cost.

Based on the principle actions of a riser recliner chair a seat raiser can help the user to go from sit to stand and vise versa with ease. For those with more complex needs a verticle lift option is also available.



Ropox

ROPOX Toilet Lifter

- Slim, modern profile
- 524mm wide x 205mm deep x 1154mm height (in lowered position)
- Utilised Quality Geberit components
- Height adjustable
 - Electric 25cm height adjustment (40 to 65cm seat height)
 - Manual 22cm height adjustment (40 to 62cm seat height)
- · Range of optional extras including support arms.





Osprey Healthcare

Toilet Buddy

- Installs Anywhere
- Free-standing unit fits over any toilet
- No assembly/tools required simply unpack
- Comes with AC plug and rechargeable battery pack
- Easy to Use
- Height Adjustable
- Stainless Steel frame for stable support
- Simple button press
- Voice activated optional extra
- Back Rest optional extra
- Raises and lowers slowly and smoothly





Drive Devilbiss Solo Toilet Lift

- Battery powered device to get on and off toilet
- Helps reduce manual handling risks for carers
- Powered lifting replicates body's natural movement
- Seat can be stopped at any position to suit user
- Set of extended feet is available as an accessory to raise seat a further 6cm (2.5") for accommodating tall users or high toilets







Aerolet (KingKraft)

Aerolet sit to stand

- Max user weight 180Kg
- Simple to use controls
- Incorporates Geberit own toilet seat for zero change to function
- Seat seal under frame ensure optimum drying







Aerolet (KingKraft)

Aerolet Vertical lifter

- Max user weight 180Kg
- Simple to use controls
- Incorporates Geberit own toilet seat for zero change to function
- Seat seal under frame ensure optimum drying







<u>Tips</u>

A Mera CARE installed with a 430mm seat height gives a Solo Toilet lifter seat height of 480mm (comfort height)

For seat heights
lower than 480mm
we recommend an
Aerolet Toilet lifter.
Available in the UK
exclusively via
KingKraft Ltd



Other equipment

There are many basic products that fit around the toilet offering various levels of support or assistance. Often non specialist equipment can be as suitable


Drop down, wall mounted arms, toilet frame surrounds, MD frames are some of the most common accessories.



Toilet surround frames

Padded Back support

- Quick and easy to install
- Can be set at height user requires
- DDA Compliant
- High quality stainless steel tube
- Concealed Fixings
- Easy clean white padded back / head rest
- Max projection: 310 mm. Max width: 542 mm
- https://www.prestigewashrooms.co.uk/prestige-stainless-steel-padded-backrest-542mm.html

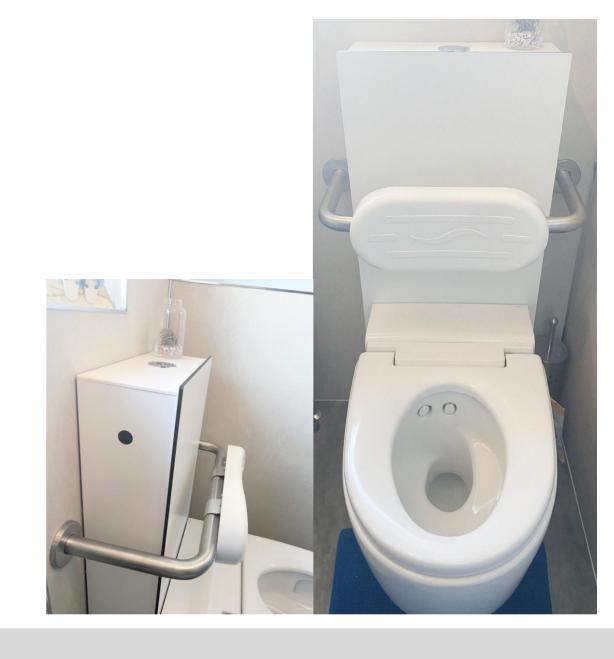





Gel Ovations

The Gel Seat

- Offers Pressure relief whilst toileting.
- Gel filled centre
- Quick and easy to remove
- Covered in waterproof Dartex material
- Non slip underside for safe transferring.







Toilet surround frames

Frames

- Quick and easy to place
- Various manufacturers
- Stock Loan store item
- Height adjustable







Wall mounted drop down rails

Frames

- Quick and easy to place
- Various manufacturers
- Available with in built toilet roll holder
- Various style and coatings







DoAbility

Easyfold Mobile Shower Commode

- Aluminium Frame
- Blue PVC & Polyurethane Upholstery
- Removable Seat & Cover for Easy Cleaning
- Soft PU Non Slip Armrest
- 4 Locking Castors
- Removable Bucket (front removal)
- Retractable Footrest
- Shower cover for seat
- Foldable for Easy Storage and transport







Daily Care


The MD Frame

- Installs Anywhere
- Free-standing unit fits over any toilet
- Screws to floor for stability
- Arm rest options
- Easy to Use
- Adjustable height arms and footplate







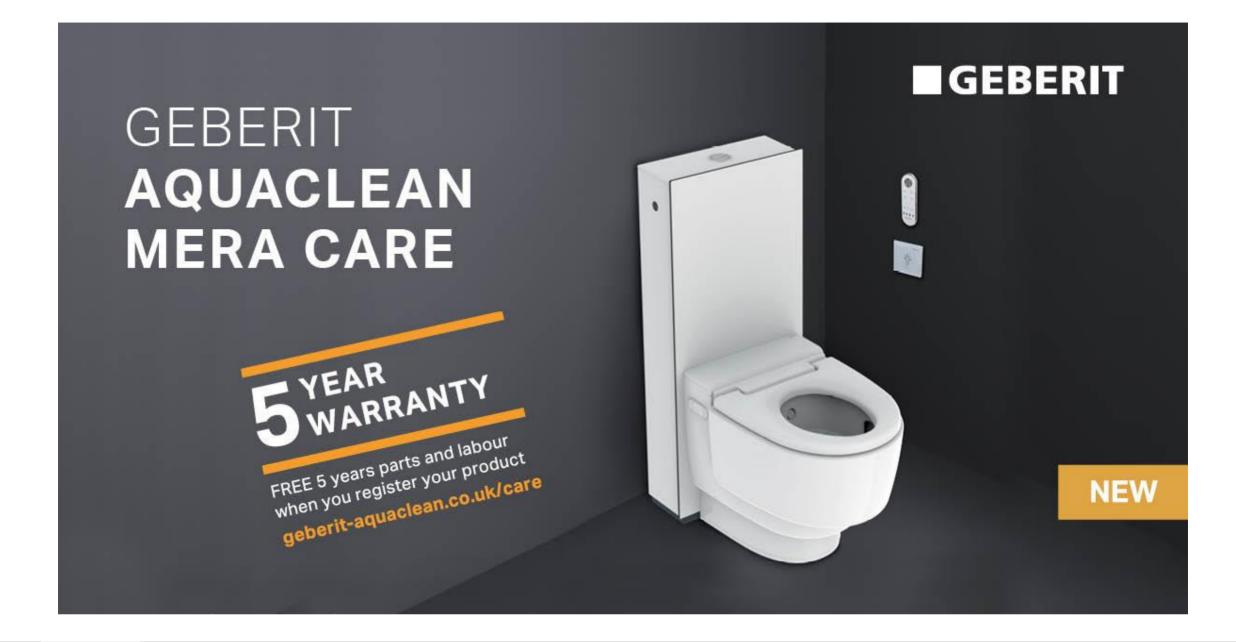



Standard Specification

- √ 5 year FREE Warranty
- √ 5 Profile settings
- ✓ Oscillating Wash
- ✓ Adjustable spray position
- ✓ Constant warm water
- ✓ Opto remote
- ✓ User remote control
- ✓ App enabled
- ✓ Lady wash
- ✓ Height adjustable
- ✓ Self servicing function

Optional Extras

- Toilet seat Lid (Product codes 243.211.11.1 & 243.404.00.1)



30 Stone User Weight Limit





Manufacturer links

www.theospreygroup.co.uk www.prismmedical.co.uk www.chilterninvadex.co.uk www.jiraffe.org.uk www.r82.com www.careandindependence.com www.kingkraft.co.uk www.dailycare.co.uk www.langham-ge.co.uk www.smarthomebathrooms.co.uk www.invacare.co.uk www.gelovationseurope.com




HOW TO SELECT ACCESSIBILITY (SHOWER CHAIR) MODE

Rear of remote control-LCD Panel and Cursors


1. Battery Compartment

2. LCD screen

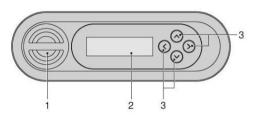


Step 1

Press Left/right cursor to launch screen, then scroll left/right until you see :

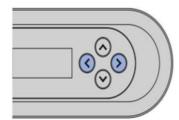


Care and Maintenance

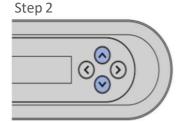

Scroll down until you see:



3. Navigation / Selection Cursors

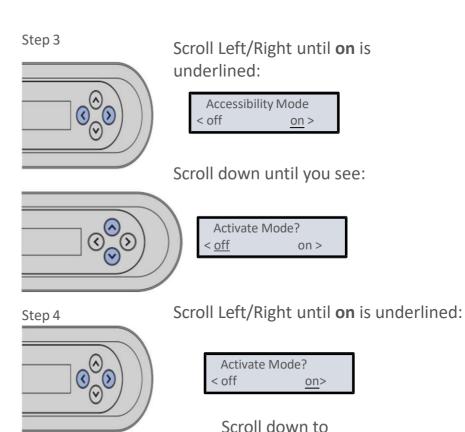

save